MSA Corporate Center  
1000 Cranberry Woods Drive  
Cranberry Twp., Pa. 16066-5207 U.S.A.  
www.MSAafety.com • 1-800-MSA-2222

315 (L) REV 4 10103562

#### ▲ WARNING

DO NOT use these respirators, filters, or cartridges for protection against exposure to radiation. They may not provide sufficient protection. Misuse can result in death or serious personal injury.

#### ▲ WARNING

This respirator/filter provides LIMITED protection. It may help reduce exposure to airborne biological agents, including H1N1 (swine) flu virus, avian (bird) flu virus, other types of influenza, SARS, or other bacterial or viral biological agents and help reduce the risk for influenza infection during a pandemic, but will NOT eliminate the risk of exposure, infection, illness, or death. • This respirator/filter is certified by NIOSH to comply with the requirements specified for the designated filter efficiency level; however, appropriate authorities have NOT established a safe level of exposure to biological agents. Therefore, the respirator may NOT prevent transmission of influenza virus. • Refer to the Centers for Disease Control and Prevention (CDC) at [www.cdc.gov](http://www.cdc.gov) for guidance on the use of respirators to help decrease

exposure to H1N1 virus or other airborne biological agents in community, home, and occupational settings. The CDC recommends fit testing, medical evaluations, and training for optimal effectiveness when a respirator is used in a non-occupational setting. Neglecting these preparatory measures may cause an unsafe condition. Respirators used in an occupational setting MUST be used in accordance with a complete respiratory protection program as required by OSHA, which includes proper selection, training, fit-testing, and fit-checking. Detailed information on a respiratory protection program is available by contacting OSHA or visiting [www.osha.gov](http://www.osha.gov). • Do NOT remove respirator in contaminated areas. The outer surface of the respirator MUST be treated as if it is

contaminated at all times. Tight-fitting safety goggles, or a full-facepiece respirator, may further help prevent transmission of influenza virus. • The CDC recommends frequent hand washing and wearing gloves to help prevent transmission of disease due to exposure to surfaces where contaminants may be present, and also immediately following removal of the respirator. • Do NOT reuse or share maintenance-free respirators. ALWAYS clean cartridge-style respirators before reuse in accordance with the instructions provided. • This respirator/filter is NOT for use by (a) children, or (b) people with a medical condition that may be adversely affected by using it. Failure to follow all warnings and instructions can result in serious personal injury or death.

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